

# CONCEPT Cassette System

Medicel CONCEPT Cassette System  
for Storz®, Millennium®, Premiere®, Protecégé® and Daisy®

## Directions for use

1. Lock the male aspiration luer adapter which is attached to the malleable flushing valve to cassette. Place other end of flushing valve into the slot on top of the cassette.
2. Insert the cassette into the cassette cavity until it seats properly.
3. Either press the IRR PRIME button or depress (and hold) the Footpedal; and insert the green striped irrigation line of the I/A tubing through the pinch valve assembly. Close the pinch valve by pressing the IRR PRIME button a second time, or by releasing the Footpedal.
4. Close the roller clamp of Irrigation Ad Set and spike and hang the irrigation bottle. Squeeze the drip chamber of the Irrigation Ad Set until the chamber is half filled with irrigation solution. Open the roller clamp and remove all air from the line. Connect the irrigation solution. Open the roller clamp and remove all air from the line. Connect the Irrigation Ad Set to the green striped irrigation line on I/A tubing set.
5. Securely thread the phaco tip onto the phaco handpiece. Thread the silicone tip cap sleeves onto the phaco and I/A handpiece until the desired orientation is achieved.
6. Connect the desired handpiece to the I/A tubing set, and remove all air from the green striped irrigation line.
7. Fill the test chamber with irrigation solution and attach to the appropriate handpiece.
8. Press ASP PRIME button or depress the Footpedal completely to remove all air from the aspiration line.
9. Hold test chamber at patient's eye level and adjust the irrigation bottle height for proper I/A balance. Consult your Operator's Manual for proper test procedures.

Please refer to the manufacturer's Operator's Manual for further instructions on the priming and tuning cycle of the Phaco Machine.

**IMPORTANT:** The Cassette Directions for Use are to be read and used in conjunction with the manufacturer's Operator's Manual. Failure to read the manufacturer's Operator's Manual could result in the improper use of this product.

If any item in this product is received in a defective condition, please notify Medicel immediately.

Manufactured by:

**Medicel AG**

Dornierstrasse 11

9423 Altenrhein

SWITZERLAND

Tel.: +41 71 727 10 50

Fax: +41 71 727 10 55

E-mail: [info@medicel.com](mailto:info@medicel.com)

Website: [www.medicel.com](http://www.medicel.com)

**CE 0482**