Instructions for use Viscoject, Viscoject-BIO, Viscoject eco

EN	Instructions for use	Page	2
DE	Gebrauchsanweisung	Seite	6
AR	تعليمات الاستخدام	صفحة	10
BG	Инструкции за употреба	страница на книгата	14
CS	Pokyny k použití	Stránka knihy	18
DA	Brugsanvisning	Bog side	22
EL	Οδηγίες χρήσης	σελίδα βιβλίου	26
ES	Instrucciones de uso	Página	30
ET	Kasutusjuhend	Raamatu leht	34
FI	Käyttöohjeet	Kirjan sivu	38
FR	Mode d'emploi	Page	42
HR	Upute za upotrebu	Stranica knjige	46
HU	Használati utasítás	Könyv oldala	50
IT	Istruzioni per l'uso	Pagina	54
KO	사용 지침	예약 페이지	58
LT	Naudojimo instrukcija	Knygos puslapis	62
LV	Lietošanas instrukcija	Grāmatas lapa	66
NL	Gebruiksaanwijzing	Pagina	70
NO	Bruksanvisning	Bokside	74
PL	Instrukcja użytkowania	Strona książki	78
PT	Instruções de utilização	Página	82
RO	Instrucţiuni de utilizare	Pagina	86
RU	Инструкция по применению	Страница книги	90
SK	Návod na použitie	Stránka knihy	94
SL	Navodila za uporabo	Stran knjige	98
SR	Uputstvo za upotrebu	Stranica knjige	102
SV	Bruksanvisning	Sida	106
TR	Kullanma talimatları	Kitap sayfası	110
UK	Інструкція з експлуатації	Сторінка книги	114

**1. Scope of application**

These instructions for use apply to the following medical devices:

Reference number	Product name
VE2200	Viscoject eco 2.2 Injector Set
VE2700	Viscoject eco 2.7 Injector Set
LP604350	Viscoject 1.8 Injector Set
LP604340	Viscoject 2.2 Injector Set
LP604335	Viscoject 2.7 Injector Set
LP604360C	Viscoject BIO 1.5 Injector Set
LP604370C	Viscoject BIO 1.6 Injector Set
LP604350C	Viscoject BIO 1.8 Injector Set
LP604340C	Viscoject BIO 2.2 Injector Set
LP604335C	Viscoject BIO 2.7 Injector Set

2. Scope

Intended use:	Folding and injecting of an intraocular lens (IOL) during eye surgery.
Indication:	Visual problems that require implantation of IOL.
Contraindications:	There are no specific contraindications for the injectors, apart from general contraindications related to ophthalmic surgery.
  User Group:	The medical device must only be handled by healthcare professionals and applied to the patient by the surgeon. Use by unqualified personnel could result in patient injury.
 Patient Group:	A patient-related, careful preoperative assessment, with well-founded clinical judgment and risk-benefit assessment, is the responsibility of the surgeon.
Clinical benefit:	IOL injector allows IOL implantation through small incisions.

3. Notes on safety

The suitability of the intraocular lens (IOL) in combination with the medical device (IOL injector) must be tested and approved in advance by the intraocular lens manufacturer.

Warning

Using a damaged or non-sterile medical device can lead to infection / endophthalmitis in the patient. Do not use the medical device if there are signs of damage to the medical device itself or to the sterile packaging.

Warning

The medical device must not be used after the expiry date. Use after the expiration date may result in infection / endophthalmitis of the patient.

Warning

Touching the endothelium with the silicone cushion or IOL can damage the corneal endothelium.

Warning

Appropriate surgical technique is the responsibility of the individual surgeon. The surgeon has to assess the suitability of the respective procedure on the basis of his / her training and experience.

Warning

The medical device is intended for single use only. Reuse or inappropriate reprocessing may lead to serious adverse effects on the health and safety of the patient.

- Reuse may result in infection / endophthalmitis, TASS or allergic reaction due to cleaning residues.

- The performance of the medical device may be compromised. Heat or the cleaning process will destroy performance properties (e.g. mechanical properties and gliding properties) of the medical device.

Caution

In order to avoid capsule rupture, the IOL must be injected slowly and carefully into the eye.

4. Reporting to manufacturer and authorities

Serious incidents occurring in connection with the device must be reported to the manufacturer and the competent authority in which the user and/or patient is established.

5. Application instructions for the single-use IOL injector

1. Open the blister in a sterile environment and remove the sterile injector (1) and cartridge (2).
2. Hold the injector so that the rear wing of the cartridge can be guided with the index finger of your left hand.
3. Fill the cartridge tip and loading chamber sufficiently with ophthalmic viscosurgical devices (OVD) (3).

It is recommended to flush the cartridge tip and the loading chamber with Balanced Salt Solution (BSS) before the application of OVD.

Allow the OVD to react (30s). OVD can lose their lubricating properties if they are in contact with air for too long. Therefore, the IOL should be injected without delay after loading.

Hydrophobic IOL types: The application of OVD is required. The exclusive use of BSS is not permitted. In addition, apply a drop of OVD to the silicone cushion (3).
4. Open the loading chamber so that the IOL can be easily loaded.


Position the IOL in the middle of the loading chamber with a concave orientation (4). Always align the IOL in the axial direction of the cartridge. If the IOL is not inserted symmetrically, it may rotate in the injector.
5. Press the IOL with sterile forceps into the two guiding rails and move the IOL back and forth slightly horizontally to ensure that it moves freely (5). Make sure that the IOL is correctly aligned and make sure that neither the edge of the optics nor the haptics is pinched when closing.

For IOLs with C-loop haptics, align the haptics with sterile forceps in the direction of the optics (6).
6. At the same time, close the wings of the loading chamber until the „Click-Lock“ mechanism engages (7).
7. Insert the cartridge into the opening in the front end of the injector. Push the cartridge into the foremost position as far as it will go (8).
8. Push the injector plunger forward.

If the silicone cushion is inclined and cannot be inserted into the closed loading chamber, the silicone cushion can be aligned with the forceps.
9. Push the injector plunger further until the compression of the inner spring begins (9).

Pull back the injector plunger a few millimetres and then push it forward again. This step ensures that the IOL is always gripped correctly.

The IOL is now loaded and ready for injection.
10. Slowly inject the IOL into the eye by applying even pressure to the injector plunger.
11. The correct exit of the IOL can be supported by turning the injector slightly. Only push the injector plunger forward until the IOL has emerged completely.
12. If necessary, assist the IOL with the help of a suitable positioning hook during the exit and bring the IOL into its final position.
13. Thoroughly remove the viscoelastic material from the eye and IOL using standard irrigation and aspiration techniques.

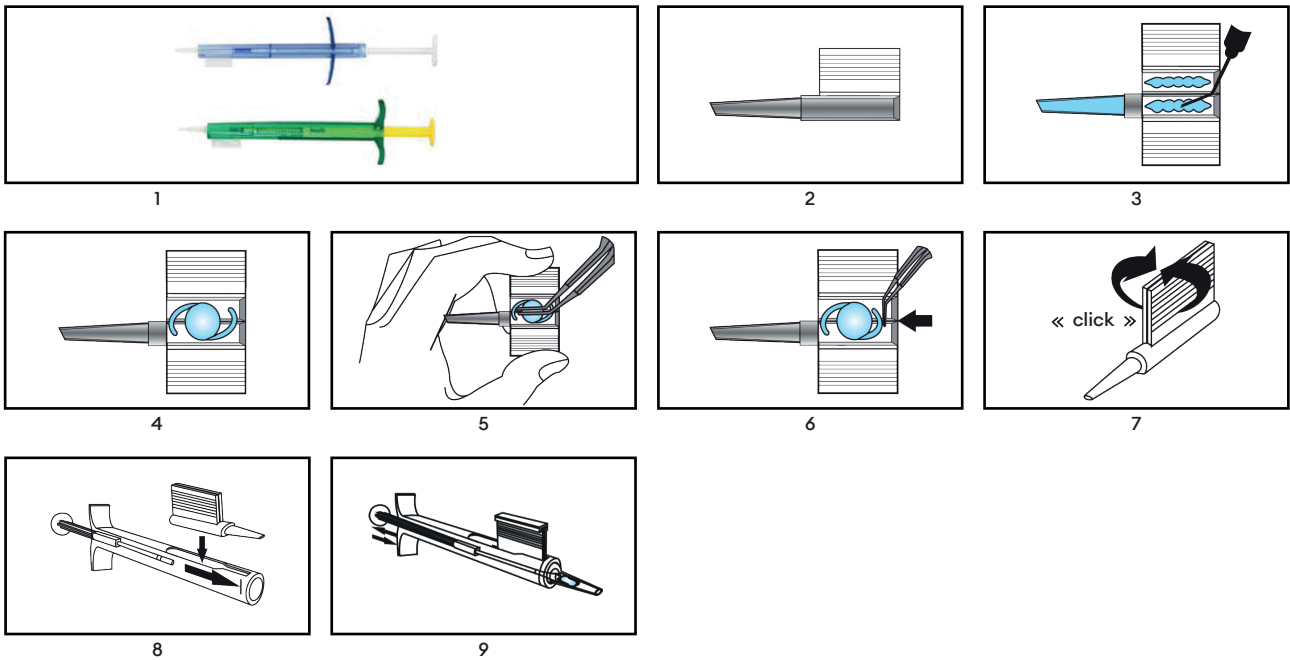6. Disposal

Warning

The medical device may be contaminated with potentially infectious substances of human origin after use. Dispose of the medical device and packaging after use according to the applicable guidelines of biological hazardous waste.

7. Symbols

	Sterilized using ethylene oxide		Keep dry
	Number of medical device in packaging unit		Do not use if package is damaged and consult instructions for use
	Single sterile barrier system		Keep away from sunlight
	Single sterile barrier system with protecting packaging outside		Do not re-sterilize
	Date of manufacture YYYY-MM-DD and country of manufacture		CE symbol with number of the notified body
	Do not re-use		Authorized representative in the European Community / European Union
	Reference number		Open here
	Lot number		Unique device identifier
	Expiry date		Medical device
	Manufacturer		Consult electronic instructions for us www.medical.com/ifu
	To be used by qualified personnel only		Caution
	Recycling		

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