NAVIJECT™
– for one-piece lenses –
– for three-piece lenses –

medicec
SWISS TECHNOLOGY FOR SURGERY

Instructions for use
Gebrauchsanweisung
Mode d'emploi
Istruzioni per l'uso
Instrucciones de uso
Kullanım Talimatı
使用说明书
Gebrauchsanwijzing

MADE IN SWITZERLAND

Manufacturer:
MEDICEC AG
Dornierstrasse 11
9423 Altenrhein
SWITZERLAND

Tel. +41 71 727 10 50
Fax +41 71 727 10 55
info@medicec.com
www.medicec.com

medicec
SWISS TECHNOLOGY FOR SURGERY

| | | | | | | | | |
|----------------|---|--|---|---|--|--|---|---|
| ENGLISH | DESCRIPTION The sterile-packed, single-use NAVIJECT™ injection system is made of high-grade materials and was designed for implanting foldable acrylic lenses. Using the "Viscoject effect", lenses can be injected into the eye safely and gently through even the smallest of incisions. The NAVIJECT™ injection system consists of three parts: 1. The sterile NAVIJECT™ single-use injector (Fig. 1) with silicone tip 2. The loading unit (Fig. 2) 3. The sterile NAVIJECT™ single-use cartridge (Fig. 3) | APPLICATION Folding and injection of one-piece foldable lenses in the capsular bag or sulcus after extra-capsular cataract extraction. INSTRUCTIONS FOR USE 1. Open the blister in the sterile area and remove the sterile NAVIJECT™ injector set. 2. NAVIJECT™ cartridges are equipped with guiding rails. This allows rapid and precise loading of the lenses. Hold the loading unit with the NAVIJECT™ cartridge in such a manner that the rear cartridge flank can be guided with the index finger of the left hand. Position the rear cartridge | flank so that the distance between the guiding rails corresponds approximately to the size of the lens optic (Fig. 4). Push the lens onto the guide rails from the rear and position it in the middle of the loading chamber (Fig. 5). 4. Apply a small amount of viscoelastic material directly to the opening of the cartridge tunnel, as well as a sufficient quantity into the loading chamber below the lens. This ensures that the "Viscoject effect" comes to bear (Fig. 6). 5. Close the cartridge on the loading unit (the lens automatically folds into the right direction) until the click-lock mechanism engages. Carefully remove the cartridge from the loading unit | (Fig. 7a, 7b). IMPORTANT: The lens should be injected within one to two minutes after loading. Viscoelastic materials may lose their lubricity if allowed to stand too long while exposed to air. 6. Guide the cartridge into the opening at the front end of the injector and push it to the very front position. Carefully press the plunger forward and guide the silicone tip into the loading chamber of the cartridge. Push the lens out of the chamber into the tapered tip of the cartridge until the tip becomes visible (Fig. 8a). IMPORTANT: Pull the plunger back a few millimetres and then | and the lens with the standard irrigation and aspiration techniques. PLEASE ENSURE: Do not re-sterilize/prepare NAVIJECT™ injections system. Single use only. Reuse and/or reesterilization may compromise device performance, which could cause serious harm to the patient's health and safety. OPERATIONAL PROCEDURE The appropriate surgical techniques are the responsibility of the respective surgeon. He or she must assess the appropriateness of the relevant procedure based on his or her education and experience. | ATTENTION: US federal regulations restrict the sale of this product to medical practitioners and those acting on their behalf.* | Batch code Use by Keep dry Keep away from sunlight Do not use if package is damaged Sterilized using Ethylene Oxide (EO) | Do not reuse Do not re-sterilize Consult instructions for use Manufacturer |
|----------------|---|--|---|---|--|--|---|---|

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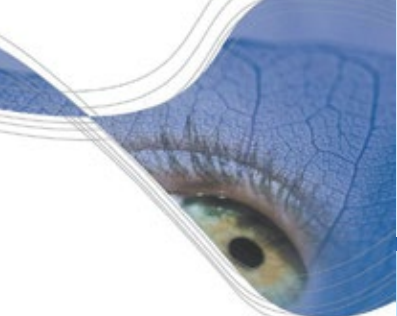
NAVIJECT™
– for one-piece lenses –
– for three-piece lenses –

medicec
SWISS TECHNOLOGY FOR SURGERY

Instructions for use
Gebruiksaanweisung
Mode d'emploi
Istruzioni per l'uso
Instrucciones de uso
Kullanım Talimatı
使用说明书
Gebruiksaanwijzing

MADE IN SWITZERLAND

Manufacturer:
MEDICEC AG
Dornierstrasse 11
9423 Altenrhein
SWITZERLAND

Tel. +41 71 727 10 50
Fax +41 71 727 10 55
info@medicec.com
www.medicec.com

medicec
SWISS TECHNOLOGY FOR SURGERY

ENGLISH

DESCRIPTION
The sterile-packed NAVIJECT™ injections system is made of high-grade materials and was designed for implanting foldable acrylic lenses. Using the "Viscoject effect", lenses can be injected into the eye safely and gently through even the smallest of incisions. The NAVIJECT™ injection system consists of three parts:

1. The sterile NAVIJECT™ single-use injector (Fig. 1).
2. The loading unit (Fig. 2)
3. The sterile NAVIJECT™ single-use cartridge (Fig. 3)

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APPLICATION
Folding and injection of three-piece foldable lenses in the capsular bag or sulcus after extra-capsular cataract extraction.
INSTRUCTIONS FOR USE
1. Open the blister in the sterile area and remove the NAVIJECT™ cartridge with loading unit.
2. NAVIJECT™ cartridges are equipped with guiding rails. This allows rapid and precise loading of the lenses. Hold the loading unit with the NAVIJECT™ cartridge in such a manner that the rear cartridge flank can be guided with the index finger of the left hand. Position the rear cartridge flap so that the distance between the guiding rails corresponds approximately to the size of the lens optic (Fig. 4).

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