ENGLISH

MediceL VISCOJECT™ easy
Single-use cartridge set
for injection of
ONE-PIECE FOLDABLE ACRYLIC LENSES

DESCRIPTION
VISCOJECT™ easy injectors – made of high-grade titanium – are instruments for implanting acrylic and silicone foldable lenses. The lens can be injected into the eye gently and safely using an injector with a VISCOJECT™ easy cartridge. The VISCOJECT™ easy injection system consists of three parts:

1. Reusable VISCOJECT™ easy titanium injector (Fig. 1a, 1b)
2. The loading unit with silicone cushion (Fig. 2)
3. The sterile VISCOJECT™ easy single-use cartridge (Fig. 3)

The cartridge may only be used once and not resterilized/treated. Reuse or resterilization may reduce the product's performance, which can result in serious impairments to the patient's health and safety. Store cartridge set at room temperature.

APPLICATION
Folding and injecting of one-piece foldable lenses into the capsular bag or sulcus after extra-capsular cataract extraction. Three-piece foldable intraocular lenses are not suited for implantation using the VISCOJECT™ easy injection system.

INSTRUCTIONS FOR USE:

1. Reuse, disinfect and sterilize the VISCOJECT™ easy injector (Fig. 1a, 1b) according to the instructions for use for the multiple-use injector "Treatment pursuant to EN ISO 17664" and place it in sterile condition in a sterile environment.
2. Open the blister package in the sterile area and remove loading unit.
3. Place the silicone tip on the injector head (Fig. 4). Make sure that the tip is upright.
4. Hold the loading unit with the VISCOJECT™ easy cartridge in such a manner that the rear cartridge flank can be guided

with the index finger of the left hand. VISCOJECT™ easy cartridges are equipped with guiding rails. Push the lens onto the guide rails from the rear and position it in the middle of the loading chamber (Fig. 5).

Ensure that the lens is correctly aligned and be careful that when closing, neither the edge of the lens nor the haptics are wedged in. For lenses with C-loop haptics, align these in the direction of the lens using steel tweezers.

5. Completely fill the cartridge tunnel of the VISCOJECT™ easy cartridge with viscoelastic solution and fill the loading chamber with sufficient viscoelastic solution (Fig. 6). In addition, apply a drop of viscoelastic solution to the silicone tip. The exclusive use of balanced salt solution (BSS) is not permitted.

IMPORTANT: When using hydrophobic lenses, let the viscoelastic solutions work for about 30 seconds to achieve sufficient hydration of the lubricating film. After 30 seconds the lens should be injected immediately. Viscoelastic materials may lose their lubricating properties when exposed to air for a longer period of time.

6. Press the cartridge sides together until the "click-lock" mechanism engages (Fig. 7).
7. Insert the cartridge into the front end of the injector. Push the cartridge into the very front position (Fig. 8).
8. Carefully push the plunger forwards and ensure that the silicone tip correctly enters the loading chamber. If the silicone tip cannot be introduced into the loading chamber, retract the plunger to the starting position and align the silicone tip straight using sterile tweezers. Continue to push the cushion until the inner spring begins to press together or until the thread completely engages (screw-type). Pull the plunger back a few millimetres and then push it forward again. This

step ensures that the lens is correctly grasped (Fig. 9).

9. Press the lens to the front part of the cartridge point outside of the eye. This prevents pressure generated by the VISCOJECT™ easy system and an increased quantity of viscoelastic solution from being transferred to the eye.
10. Insert the cartridge tip through the incision and slide it over the lens at the proximal edge of the pupil.
11. Slowly press the plunger forward (the lens is pressed forward almost without contact as a result of the pressure created by VISCOJECT™ easy). Depending on the lens model, support the correct ejection of the lens by slightly turning the injector. Slowly inject the lens into the eye and simultaneously withdraw the instrument from the eye. To avoid swelling of the silicone tip during ejection from the cartridge, only push the plunger until the lens has emerged completely, even if the plunger is not yet at the limit.

12. Carefully position the lens, rotating it if necessary with the aid of using a suitable positioning hook.

13. Remove the viscoelastic material thoroughly from the eye and the lens with the standard irrigation and aspiration techniques.
14. The VISCOJECT™ easy injector can be reused after thorough cleaning, disinfection and sterilization.

REUSABILITY
The instruments can be reused as long as they are handled carefully and are undamaged and uncontaminated. The user takes responsibility for the use of damaged or contaminated instruments. Any liability is excluded if the instruments are not used as intended.

PLEASE NOTE:
The injector's cartridge may not be treated or resterilized. The cartridge may only be used once. Information concerning the

use of the injector is contained in the instructions for use of the VISCOJECT™ easy injector set.

OPERATIONAL PROCEDURE
The appropriate surgical techniques are the responsibility of the respective surgeon. He or she must assess the appropriateness of the relevant procedure based on his or her education and experience.

GUARANTEE AND LIMITATION OF LIABILITY
The manufacturer guarantees that this product was produced with appropriate care and shall assume no responsibility for incidental or consequential damages, losses or costs that should result directly or indirectly from the use of this product. Liability is solely limited to claim-related repairs that must be performed on the product which are clearly not attributed to incorrect handling or the use of lenses not validated for this injector model.

ATTENTION: US federal regulations restrict the sale of this product to medical practitioners and those acting on their behalf.*

	Batch code		Do not reuse
	Use by		Do not resterilize
	Keep dry		Consult instructions for use
	Keep away from sunlight		Manufacturer
	Do not use if package is damaged		
	Sterilized using Ethylene Oxide (EO)		

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